



# space station

two tier back table

**CDS-3060** Dimensions  
 Main Table 30"D x 60"W x 34"H  
 Top Shelf 20"D x 60"W



**CDS-3072** Dimensions  
 Main Table 30"D x 72"W x 34"H  
 Top Shelf 20"D x 72"W

A specialty back table designed with additional room for large cases such as orthopedics, spinal fusions, neuro and craniotomies. The tuck-away cantilevered shelf holds multiple trays and is angled for clear observation of instruments.

The Space Station is easily rolled into position using 4" diameter, heavy-duty ball bearing brake/swivel casters. To lock swivel caster, simply engage the single-pedal brake.

## One Person Operation



Pneumatic assist enables one-person operation in raising or lowering the cantilevered shelf.



Shelf is easily raised when extra space is required, or lowered into storage position if not needed.



Cantilevered shelf features a 10° slope and has an adjustable height from 49.9" to 52.9".

**trusted fit**   
 two-tier drapes

## Disposable Table Drapes

Pedigo Trusted Fit drapes feature heavy-duty three layer construction. 12 Drapes per case.

**CDS-3060-DD** 5' table (1.5m)

**CDS-3072-DD** 6' table (1.8m)





# space station

electric adjustable back table

Pedigo's **NEW** Space Station back table is now electro-hydraulically height adjustable and LED illuminated for improved visibility.

efficiency • ergonomic • safety

For patient and staff safety, the Space Station Two-Tier Back Table meets the highest hospital safety standards. IEC 60601-1 and UL approved E 503 186 approved.



- Elevate & Illuminate Table
- CDS-3060-ADJ-E ....5' table (1.5m)
- CDS-3072-ADJ-E ....6' table(1.8m)
- Illuminate Table
- CDS-3060-L.....5' table (1.5m)
- CDS-3072-L.....6' table(1.8m)



Power Supply

Height Adjustment

The UP/DOWN buttons for height control are located to the left of the power supply.

Cord Loops

Power Cord

Hospital grade power cord can be wound around cord loops when not in use. Cord length measures 9' 8" (3m).

Latch Release

Pull up on the release latch to disengage and lower upper tier. Release Latches are located on either side of the table.



**Casters**

The Electric Adjustable Space Station features four Total Lock Casters each measuring 4" (10cm) in diameter.



**Power Supply**

At the back of the table are two independent ON/OFF toggle switches to control the entire table unit and a second for the LED lights.



draping  
**Space Station**  
video demo

draping video demo online  
[www.pedigo-usa.com](http://www.pedigo-usa.com)

**new!**



# space station

electric adjustable back table

Pedigo's **NEW** Space Station back table is now electro-hydraulically height adjustable and LED illuminated for improved visibility.

**efficiency • ergonomic • safety**

For patient and staff safety, the Space Station Two-Tier Back Table meets the highest hospital safety standards. IEC 60601-1 and UL approved E 503 186 approved.



- Elevate & Illuminate Table
- CDS-3060-ADJ-E ... 5' table (1.5m)
- CDS-3072-ADJ-E ... 6' table(1.8m)



- Illuminate Table
- CDS-3060-L ..... 5' table (1.5m)
- CDS-3072-L ..... 6' table(1.8m)



Upper tier in storage position

Upper tier being raised

Upper tier in raised position

